RC-FIT® OPEP

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△ WARNING

Please clean before the first use! Please ensure that the RC-FIT® OPEP is being used in accordance with these instructions for use. Its incorrect use (off-label-use) will result in an exclusion of liability for the manufacturer.

IMPORTANT INFORMATION

Before using the RC-FIT® OPEP for the first time, please read the Instructions for Use carefully. If you have additional questions regarding its usage, please consult with your qualified medical practitioner. The RC-FIT® OPEP is well-suited for users starting at 3 years of age in home and clinical setting. The patient must be able to purposefully coordinate inhalation and exhalation through the mouth. Children should only undergo training with the assistance of an adult. In the case of severely impaired patients, the training should be conducted together with a nurse or physiotherapist. Paraplegic patients or persons who suffer from neuromuscular diseases should only undergo training on the advice of qualified medical practitioners. Please abstain from any food intake shortly before and during the training.

Action to be taken in the event of a severe coughing fit: Following intake of your medication, try to work slowly with the respective RC-FIT® OPEP in its initial setting, in order to also dilate the bronchi mechanically.

Important: All individual parts must be stored out of the reach of children.

After every use, please allow the respective RC-FIT® OPEP to dry as moisture may form in the product with long-term training.

USE OF OSCILLATING EXHALATION THERAPY:

- 3 x daily
- with prolonged exhalation for approx. 10–30 breaths each time
- exhalation only through the mouth

The RC-FIT® OPEP is a unique, sound-generating/oscillating, expiratory relaxation and PEP therapy device. Whether used for prevention, rehabilitation or targeted improvement of fitness or pure relaxation – the effect of the RC-FIT® OPEP is immediate and noticeable all over.

SCOPE OF DELIVERY (MATERIAL COMPOSITION)

RC-FIT® OPEP	
Mouthpiece	MOUTHPIECE RED with 4 intensity levels [in mm: 6/5/4/3,75] (polypropylene)
Inlay with integrated exhalation valve	Transparent (polypropylene/rust-free stainless steel/Nickel silver)
Intensity regulator	Grey (polypropylene)
Deckel	grau (Polypropylen)

△ CONTRAINDICATIONS

Therapy with the RC-FIT® OPEP produces an overpressure in the lungs during exhalation. Therefore, contraindications for its use are:

- Presence of pneumothorax, blood-expectorating cough (massive haemoptysis), and cranial pressure symptoms > 20 mmHg.
- Decompensated cardiovascular diseases
- Acute thromboses
- Elevated coagulation factor
- Acute blood-expectorating cough (haemoptysis)
- Possible eardrum injuries
- Pulmonary barotrauma
- Significant cardiovascular insufficiency
- Massive restriction of the respiratory pump
- Skin emphysema
- Severe infection with fever

The following relative contraindications should be discussed with your treating healthcare professional:

- Tracheoesophageal fistula
- Bronchial asthma
- Pulmonary emphysema
- Sinus infection
- Repeated, substantial nosebleeds
- Granulo ma - growths in the mucous membranes near the tracheostoma after surgery or injuries to the mouth, face, skull, or oesophagus

Note: Training/therapy following an eye operation (cataracts or glaucoma) should only start beginning the 4th week post-surgery.

SIDE EFFECTS

An exhalation pressure of more than 20 cmH₂O can cause patients sensitive to increased transpulmonary pressure to experience one or more of the listed side effects:

- Hyperventilation (excessive increase in respiration) may occasionally cause slight vertigo or numbness in the mouth and hands. These symptoms are not dangerous and diminish once the respiration has normalized.
- Hypoventilation with hypercapnia (elevated CO₂ content).

Other possible side effects

- Elevated intracranial pressure
- Cardiovascular risk
- > Insufficient oxygen supply to the heart
- > Decreased venous return
- Swallowing of air with the possibility of vomiting and aspiration of vomit

PURPOSE OF USE

The RC-FIT® OPEP trains the strength and endurance of the exhalation muscles, holds the airways open, and loosens stubborn secretions in the case of acute and chronic diseases of the lower respiratory tract.

The RC-FIT® OPEP can effectively train the pursed lips breathing technique. This training of the respiratory tract holds the airways open during exhalation using resistance and vibrations. In this manner, a bronchial collapse is prevented and phlegm dissolution is activated.

Additionally, the RC-FIT® OPEP is a training device and an aid in emergency cases.

Duration of use

If used daily, we recommend the exchange of the RC-FIT® after one year. Please notify the manufacturer of all serious incidents occurring in connection with the device.

Storage

Store the RC-FIT® in dry condition and protect the assistive device from sunlight and dust, among other things.

Disposal

The RC-FIT® can be disposed of with household waste, as long as not prescribed otherwise by the respective country's disposal regulations.

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Average pressure during exhalation

intensity level	Flow rate (pressure difference to ambient pressure)			
	10 l/min	15 l/min	20 l/min	25 l/min
I	1,5 cmH ₂ O	1,7 cmH ₂ O	1,9 cmH ₂ O	2,1 cmH ₂ O
II	2,1 cmH ₂ O	2,6 cmH ₂ O	2,8 cmH ₂ O	3,1 cmH ₂ O
III	16,4 cmH ₂ O	17,3 cmH ₂ O	78,3 cmH ₂ O	89,5 cmH ₂ O
IV	38,7 cmH ₂ O	64,4 cmH ₂ O	79,8 cmH ₂ O	95 cmH ₂ O

Frequency during exhalation: 130 Hz+/-5%

- III. A) Place the RC-FIT® OPEP in your mouth and close your lips around the mouthpiece. Please do not close the triangle with your fingers.

Breathe in through the nose and out through the mouth.

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Is the training too light, or the sound is too loud even after 15 breaths, increase the intensity (resistance) by rotating the mouthpiece, see Fig. II.

CLEANING AND DISINFECTION (Boiling)

If used regularly, clean and disinfect the device once a week, please clean daily if there is a lot of mucus. If necessary (e.g. if no more sounds are produced), insert parts of the RC-FIT® OPEP in a warm, light detergent solution and then boil out.

1. Cleaning and disinfection

For cleaning purposes, pull off the mouthpiece from the intensity regulator. Then, carefully remove the inlay with integrated valve from the intensity regulator.

IMPORTANT: Try to avoid any contact with the integrated valve!

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2. Drying

In order to prevent mould formation, all parts must be completely dried. To accelerate this process, shake off any excess water before setting the parts down. Allow all parts to air dry. Do not rub dry the inlay with towels.

3. Assembly

Assemble the device after washing your hands and ensure that all parts of the RC-FIT® OPEP are clean. There may be no dust residue or scattered particle deposits, as these could be breathed in and reach the lungs.

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The proprietor of the trademarks and registered trademarks is the company CEGLA Medizintechnik GmbH.