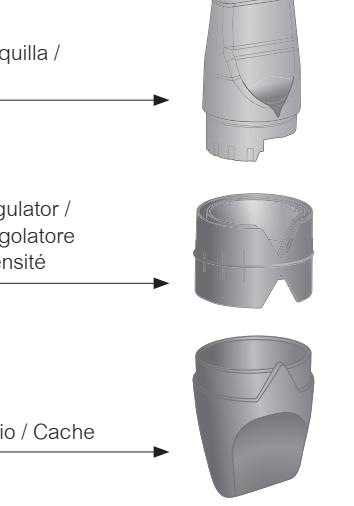
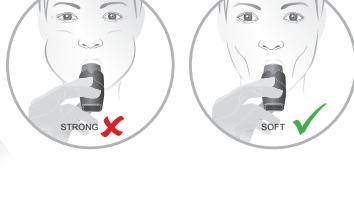
RC-FIT® PEP/IMT

Mundstück / Mouthpiece / Boquilla / Boccaglio / Embout buccal

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△ WARNING

Please clean before the first use! Please ensure that the RC-FIT® PEP/IMT is being used in accordance with these Instructions for Use. Its incorrect use (off-label-use) will result in an exclusion of liability for the manufacturer.

IMPORTANT INFORMATION

Before using the RC-FIT® PEP/IMT for the first time, please read the instructions for Use carefully. If you have additional questions regarding its usage, please consult with your qualified medical practitioner. The RC-FIT® PEP/IMT is well-suited for users starting at 3 years of age in home and clinical setting. The patient must be able to purposefully coordinate inhalation and exhalation through the mouth. Children should only undergo training/therapy with the assistance of an adult. In the case of severely impaired patients, the training/therapy should be conducted together with a nurse or physiotherapist. Paraplegic patients or persons who suffer from neuromuscular diseases should only undergo training/therapy on the advice of qualified medical practitioners. Please abstain from any food intake shortly before and during the training/therapy.

Action to be taken in the event of a severe coughing fit: Following intake of your medication, try to work slowly with the respective RC-FIT® PEP/IMT in its initial setting, in order to also dilate the bronchi mechanically.

Important: All individual parts must be stored out of the reach of children. Training/therapy following an eyeoperation (cataracts or glaucoma) should only start beginning the 4th week post-surgery. After every use, please allow the respective RC-FIT® PEP/IMT to dry as moisture may form in the product with long-term training/therapy.

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For the PEP/IMT combination, please inhale and exhale through the mouth!

SCOPE OF DELIVERY (MATERIAL COMPOSITION)

RC-FIT® PEP/IMT

Mouthpiece	MOUTHPIECE TRANSPARENT with 7 intensity levels [in mm: 4/3,75/3,5/3,25/3,2/2] (polypropylene)
Intensity regulator	grey (polypropylene)

Deckel

Lid

USE OF IMT INHALATION TRAINING:

- several times a day and as needed for approx. 20 breaths each time
- exhalation only through the mouth

For the PEP/IMT combination, please inhale and exhale through the mouth!

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RC-FIT® PEP/IMT

A) TRAINING & THERAPY

The same principle applies to sports and everyday life: whoever has more oxygen available has a greater ability to function.
By only breathing through the RC-FIT® PEP/IMT, the respiratory muscles can be trained. By also breathing out through the RC-FIT® PEP/IMT, the airways can be expanded.

B) ANCHOR FOR SHORTNESS OF BREATH

As with the pursed lips technique, the RC-FIT® PEP/IMT can stabilise the bronchi in cases of shortness of breath, through a deliberately slow exhalation. Hyperventilation can be reduced or prevented. Thanks to its practical trouser pocket shape, the device is always close at hand in the lowest intensity level.

USE

The use of the RC-FIT® PEP/IMT is not dependent on the force of gravity. The device may thus be used in a reclining, sitting or standing position. For the best effect, however, using the device while standing is preferable. Drink a glass of water before training in order to keep your airways moist.

- I. Please inspect the device for foreign bodies or damages prior to every use. If any parts are missing or damaged, replace them immediately.
- II. Start your first training session in the lowest intensity level "MINIMUM" (largest possible triangle). The smaller the triangle becomes, the higher your training and therapy load (intensity). The RC-FIT® PEP/IMT has a total of 7 intensity levels.

Average pressure during exhalation / inhalation

intensity level	Flow rate (pressure difference to ambient pressure)
I	10/-10 l/min 0,9/-1,7 cmH ₂ O 2/2,3 cmH ₂ O 3,6/-4,1 cmH ₂ O 5,5/-6,4 cmH ₂ O
II	1,1/-1,3 cmH ₂ O 2,5/-2,7 cmH ₂ O 4,4/-4,8 cmH ₂ O 6,6/-7,5 cmH ₂ O
III	1,4/-1,5 cmH ₂ O 2,9/-3,4 cmH ₂ O 5/5,8 cmH ₂ O 7,7/-8,9 cmH ₂ O
IV	1,9/-1,9 cmH ₂ O 4,1/-4,2 cmH ₂ O 7,1/-7,2 cmH ₂ O 10,9/-11,1 cmH ₂ O
V	2,2/-2,2 cmH ₂ O 4,6/-4,6 cmH ₂ O 8,1/-8,1 cmH ₂ O 12,3/-12,3 cmH ₂ O
VI	3,2/-3,2 cmH ₂ O 7,2/-6,9 cmH ₂ O 12,3/-11,7 cmH ₂ O 18,7/-17,9 cmH ₂ O
VII	5,3/-5,6 cmH ₂ O 11,5/-11,8 cmH ₂ O 19,4/-20,3 cmH ₂ O 27,7/-30,1 cmH ₂ O

- III. A) Place the RC-FIT® PEP/IMT in your mouth and close your lips around the mouthpiece. Please do not close the triangle with your fingers.

PEP: Inhalation through the nose and exhalation only through the mouth
IMT: Inhalation only through the mouth and exhalation through the nose or inhalation and exhalation via the RC-FIT®

A) Depending on your training condition, take a 1-minute break after 1½ minutes. One training unit should consist of 10-30 breaths 3 x daily. In the case of Parkinson's with dysphagia, training of 5 x daily with 5 exhalations each time should be conducted.

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If training feels too easy, increase the intensity (resistance) by rotating the mouthpiece, see Fig. II.

CLEANING AND DISINFECTION (Boiling)

If used regularly, clean and disinfect the device once a week, please clean daily if there is a lot of mucus.

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For cleaning purposes, pull off the mouthpiece from the intensity regulator. Pour all components into lukewarm clean water and standard liquid soap detergent. Rinse with clear water. After this cleaning, disinfect all parts in a vaporizer or in boiling water for approx. 5 minutes.

2. Trocknen
In order to prevent mould formation, all parts must be completely dried. To accelerate this process, shake off any excess water before setting the parts down. Allow all parts to air dry.

3. Assembly

Before assembly, please wash your hands and ensure that all parts of the RC-FIT® PEP/IMT are clean. There may be no dust residue or scattered particle deposits, as these could be breathed in and reach the lungs.

FREQUENCY OF USE

Recommended therapy intervals for the respiration:
PEP: 2-3 x daily with prolonged exhalation for approx. 20 breaths each time
IMT: several times a day and as needed, approx. 1-3 minutes

TIPS:

- Integrate the therapy and training into your daily routine, especially when you notice very little mucus, e.g. always before brushing your teeth.
- To obtain a better effect from your medication spray, integrate this inhalation into therapy with the RC-FIT® PEP/IMT.

ACCESSORIES

The RC-Mask Tracheostoma allows tracheotomised patients to undergo respiratory therapy and respiratory training with the RC-FIT® PEP/IMT. The RC-Mask Tracheostoma is simply placed over the mouthpiece of the RC-FIT® model and then connected with the open tracheostomy, i.e. the cannula.

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The proprietor of the trademarks and registered trademarks is the company CEGLA Medizintechnik GmbH.