
RC-Cornet® PLUS NEBULIZER KIT

The RC-Cornet® PLUS NEBULIZER KIT enables a combination of the RC-Cornet® PLUS variants with almost all non standard nebulizers.*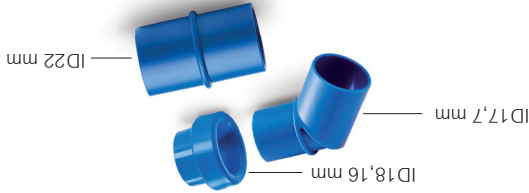

This RC-Cornet® NEBULIZER KIT package contains: Nebulizer, elbow connector and RC-adapter


Article no. 2651 | PPN 111 443 990 049

* Depending on the application deviations in the deposition data are possible.



Please read the instructions for use RC-Cornet® PLUS NEBULIZER KIT carefully before first use. Please also follow the instructions of the according RC-Cornet® PLUS respiratory therapy devices, the nebulizer and the medication. All three products are not included and must be purchased additionally. In combination with nebulizers, it is important to ensure that the RC-Cornet® PLUS with and without the RC-Cornet® PLUS NEBULIZER SET is as securely fitted as it would be if the nebulizer mouthpiece was attached. During inhalation you should not see any inhalation solution flowing outside of the product combination. To this end, pay attention to the respective interior diameter (ID).

All trademarks and registered trademarks belong to the company CEGLA Medizintechnik GmbH, Germany. In some instances, product graphics may vary from actual product models, sizes and/or colors listed below.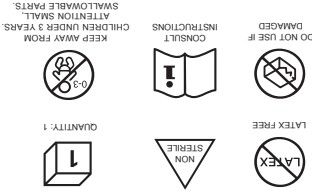

CEGLA
Medizintechnik GmbH
Horresser Berg 1
56410 Montabaur
Germany
Tel +49 2602 92 13-0
info@cegla.de
www.cegla.de

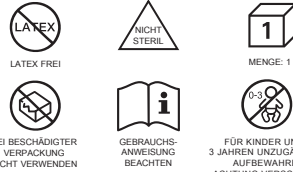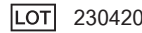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
Material: Polypropylen

Remove the nebulizer mouthpiece. Then connect the nebulizer with
one, two or all three parts of the RC-Cornet® PLUS NEBULIZER KIT
(see illustrations above).
Now apply the RC-Cornet® PLUS variants (RC-Cornet® PLUS,
RC-Cornet® PLUS NASAL, RC-Cornet® PLUS TRACHEO) according
to the characteristics in setting 1.
The integrated one way valve now enables an inhalation of
aerosol once you have started your compressor. When you exhale,
the valve closes and you start your respiratory physiotherapy via
the RC-Cornet® PLUS variants.
Please be aware that the nebulizer as mentioned in the instructions
for use is always in an upright position.
TIPP: If you feel uneasy to use setting 1 at the beginning, you can
use the middle setting between 1 and 2 to start with.
Recommended usage:
The timing should be equivalent to the recommended timing going
with the nebulizer or the drugs applied.
The same cleaning instructions apply for the RC-Cornet® PLUS
NEBULIZER KIT as for the corresponding RC-Cornet® PLUS product.
Material: Polypropylen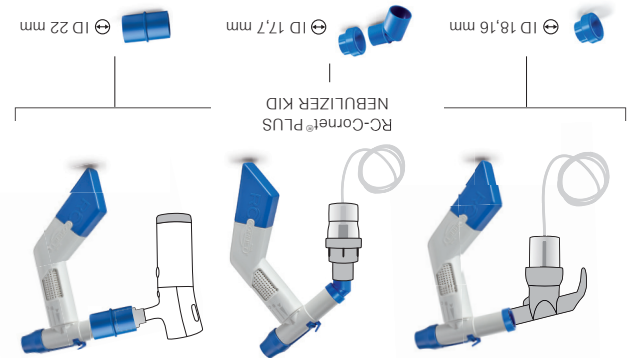

COMBINED THERAPY / NEBULIZER CONFIGURATION
WITH THE RC-Cornet® PLUS VARIANTS: