DEUTSCH

1 ANWENDUNG

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ENGLISH

1 USE

I Please inspect the device for foreign bodies or damages prior to use. If any parts are missing or damaged, replace them immediately.

Settings: RC-COMET® PLUS can be used for two different forms of treatment. Select the treatment options using the settings I or II on the mouthpiece. Change the settings by turning the mouthpiece. Please turn the mouthpiece to the stop. Unless otherwise prescribed, use both settings for at least two minutes each. Always start therapy with setting I.

• Setting I: stands for a stabilizing positive expiratory pressure with oscillations (OEP) as well as with and without inhalation – This widens the airways and reduces breathlessness.

• Setting II: Mobilising expiratory pressure – free the airways from thick bronchial mucus. This makes coughing more productive. Once the airways have been cleared, the urge to cough automatically diminishes.

III Hold the device as shown. Insert the RC-COMET® PLUS into the mouth and close the lips around it to ensure an effective seal. Please do not bite on the mouthpiece or block the nebulizer port your hand. Breathe in through nose or mouth and after a brief pause, exhale through RC-COMET® PLUS at a normal rate and intensity.

IV Exiting into RC-COMET® PLUS produces a positive pressure in the lower respiratory tract as well as vibrations. As you become used to breathing out through the RC-COMET® PLUS, take deeper breaths and exhale after a brief pause all of the air through the RC-COMET® PLUS. Ideally the exhalation phase should take at least 3-4 times longer than inhalation. TIP: Use your free hand to press your cheeks together to avoid a loss in pressure and to maximise treatment effectiveness.

V In addition to the settings described under II, 4 different levels of resistance can be selected for each setting. This allows an individual patient suitable training by choosing the right level of resistance (outlet pressure). Start with level A and increase the levels for each condition. Set the device by turning the body against the adapter. Please verify the therapy with your healthcare professional.

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Clinical recycling instructions can be found at www.cegla.de

Important! The cleaning aid must remain in the tube during the entire cleaning process.

If used regularly, clean the device at least twice weekly.

Further cleaning options with the RC-Clean® cleaning bag.

Wash all parts in a solution of lukewarm clean water and standard liquid soap detergent. Rinse with clear water. After this cleaning, all parts may be cleaned in a vaporiser or for approx. 5 minutes in boiling water. Maximum temperature 134 °C.

Combining RC-COMET® PLUS therapy with a standard ISO nebulizer can improve delivery of your nebulized medication and reduces the total treatment duration. Furthermore the effectiveness of both therapies can improve.

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