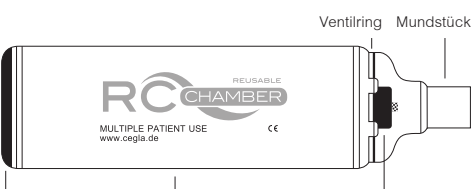
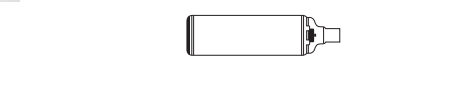
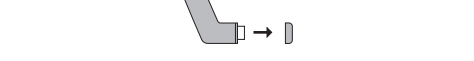
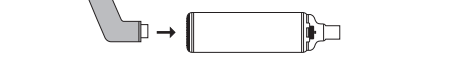
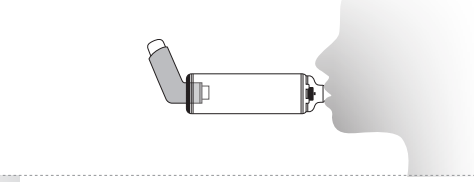
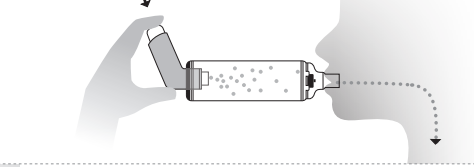
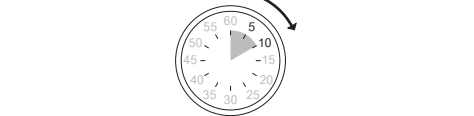
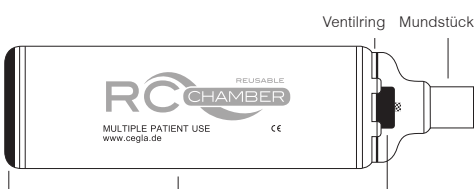
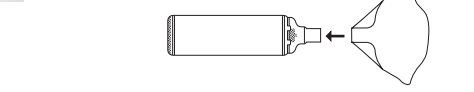
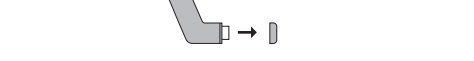
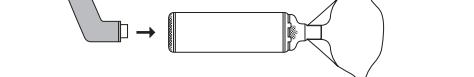
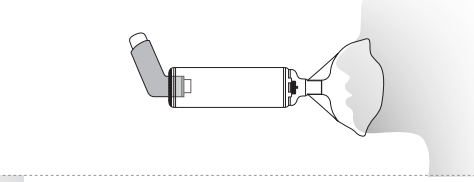
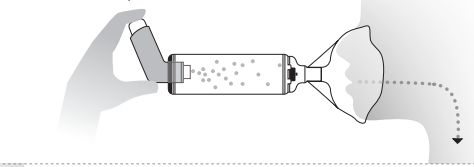
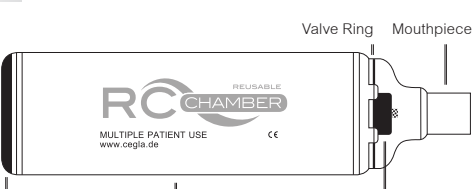
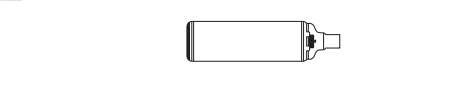
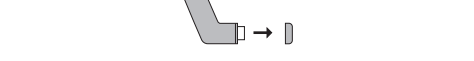
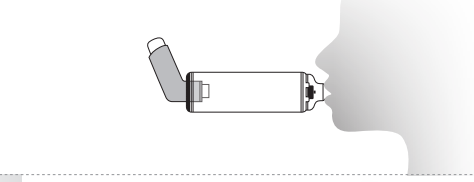
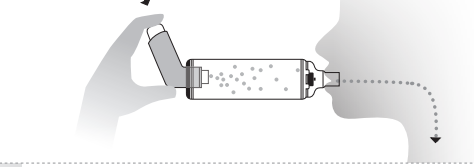
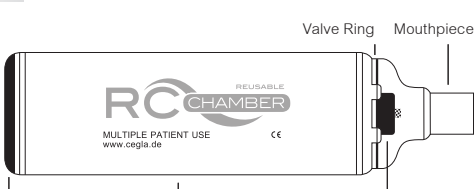
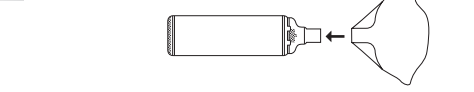
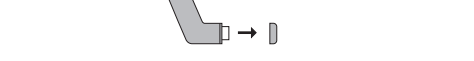
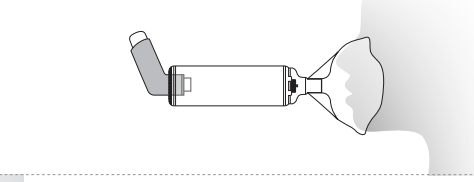
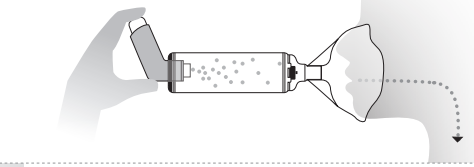


KURZANLEITUNG mit Mundstück I  II  III  IV  V  VI  VII 	KURZANLEITUNG mit Maske I  II  III  IV  V  VI  VII 	ANWENDUNGSBESCHREIBUNG I Prüfen Sie stets, ob sich im RC-Chamber® reusable Fremdkörper befinden, Teile beschädigt sind oder fehlen. II Für die Anwendung mit Maske: Stecken Sie die Maske auf das Mundstück des RC-Chamber® reusable. III Die Verschlusskappe vom Mundstück des Dosieraerosols entfernen und das Dosieraerosol vor jedem Gebrauch kräftig schütteln. IV Das Mundstück des Dosieraerosols gemäß Abbildung in den vorgesehenen Gummiadapter des RC-Chamber® reusable stecken. V Das Mundstück des RC-Chamber® reusable mit den Lippen fest umschließen bzw. die Maske dicht auf Nase und Mund setzen. VI Die Kartusche des Dosieraerosols nach unten drücken, um einen Sprühstoß in die Kammer des RC-Chamber® reusable abzugeben. VII Tief einatmen und den Atem für 5–10 Sek. anhalten, um die therapeutische Wirkung zu optimieren. Kinder unter 18 Monaten sollten 4–6 Atemzüge, Kinder und Erwachsene 2–4 Atemzüge inhalieren. Sind mehrere Sprühstöße verordnet worden, sollte eine Minute vergehen, ehe die Schritte V bis VII wiederholt werden!	VERWENDUNGSZWECK Der RC-Chamber® reusable ist eine Inhalationshilfe, die speziell für die Therapie mit Medikamentensprays – sogenannte Dosieraerosole – entwickelt wurde und zur Behandlung von Atemwegserkrankungen eingesetzt wird. Der flexible Gummiadapter ermöglicht eine Therapie mit allen gängigen Dosieraerosolen. Mithilfe des RC-Chamber® reusable entfällt die Koordination zwischen Auslösen des Dosieraerosols und Inhalation des Medikamentes. Außerdem werden die großen, nicht lungengängigen Partikel ausgefiltert und verhindern so die Reizung sowie den Pilzbefall im Mund- und Rachenraum. ERHÄLTICHE PRODUKTE FÜR DIE KLINIK RC-Chamber® reusable 230 ml • mit Mundstück: VE 20 Artikel 3061-20 • mit Säuglingsmaske: VE 10 Artikel 3071-10 • mit Kindermaske: VE 10 Artikel 3081-10 • mit Maske für Erwachsene sowie ältere Kinder: VE 10 Artikel 3091-10 Säuglingsmaske: VE 10 Artikel 30110-10 Kindermaske: VE 10 Artikel 30210-10 Maske für Erwachsene sowie ältere Kinder: VE 10 Artikel 30410-10 RC-Maske Tracheostoma: VE 10 Artikel 3012-10 
QUICK START GUIDE with mouthpiece I  II  III  IV  V  VI  VII 	QUICK START GUIDE with mask I  II  III  IV  V  VI  VII 	INSTRUCTIONS FOR USE I Continually check for foreign objects in the RC-Chamber® reusable, for damage to any of the parts or if any parts are missing. II For mask use: Place the mask on the RC-Chamber® reusable mouthpiece. III Remove the sealing cap from the mouthpiece of the dose inhaler and shake dose inhaler spray vigorously before each use. IV Place the mouthpiece of the dose inhaler as shown in the figure in the provided rubber adapter of the RC-Chamber® reusable. V Place your lips tightly around the mouthpiece or place the mask airtight over the nose and mouth. VI Push the cartridge of the dose inhaler down in order to dispense a spray into the chamber of the RC-Chamber® reusable. VII Breathe in and hold the breath for 5 to 10 seconds to optimize the effect. Children under 18 months should inhale 4 to 6 breaths, children and adults 2 to 4 breaths. Should several sprays be prescribed, then a one minute break should be taken before repeating steps V to VII!	PURPOSE FOR USE The RC-Chamber® reusable is an inhalation device which was especially developed for MDI therapy with metered dose inhalers – so-called dose inhaler – and can be used for treatment of respiratory illnesses. The flexible rubber adapter is compatible with all common dose inhaler. With the help of the RC-Chamber® reusable the coordination between releasing the dose inhaler and inhalation of the medicine is omitted. In addition the large, not respirable particles are filtered out. Therefore irritation as well as a fungal attack in the mouth and throat is prevented. AVAILABLE PRODUCTS FOR THE CLINIC RC-Chamber® reusable 230 ml • With mouthpiece: 20 pcs. article-no. 3061-20 • With infant mask: 10 pcs. article-no. 3071-10 • With child mask: 10 pcs. article-no. 3081-10 • With mask for adults as well as older children: 10 pcs. article-no. 3091-10 Infant mask: 10 pcs. article-no. 30110-10 Child mask: 10 pcs. article-no. 30210-10 Mask for adults and older children: 10 pcs. article-no. 30410-10 RC-Tracheal mask: 10 pcs. article-no. 3012-10 