RC-CORNET[®] ADAPTER WITH MOUTHPIECE

To combine the respiratory therapy and inhalation therapy with the RC-Cor-net[®] and the RC-Cor-net[®] N

Please read the RC-Cor-net[®] adapter with mouth-piece instructions for use carefully before first use. Please also follow the instructions of the nebulizer (the nebulizer is not included).


In combination with nebulizers, it is important to ensure that the RC-Cor-net[®] is as securely fitted as it would be if the nebulizer mouthpiece were attached. In other words, during inhalation you should not see any inhalation solution flowing outside of the product combination. To this end, pay attention to the res-pective interior diameter (ID 20,3mm). Also, please read the instructions for use of your nebulizer and your inhalation device, as well as of the corres-ponding medication. Depending on the application devi-ations in the deposition data are possible.

CEGLA
MEDIZINTECHNIK

CEGLA Medizintechnik
GmbH & Co. KG
Horresser Berg 1
56410 Montabaur
Germany
Tel +49 2602 9213-0
info@cegl.de
www.cegl.de

RC-Cor-net[®] adapter with mouthpiece (ID 20,3 mm)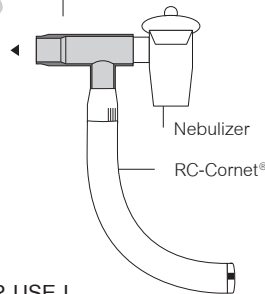

INSTRUCTION FOR USE I

For the combination therapy of the lower respiratory tracts


The RC-Cor-net[®] adapter with mouthpiece combines the RC-Cor-net[®] with all common nebulizers.

Remove the mouthpiece of the nebulizer first. Now connect the RC-Cor-net[®] adapter with mouthpiece to the nebulizer (see image). Caution: The mouthpiece of the nebulizer must first be removed.

Then slide the adapter onto the RC-Cor-net[®] so that it fits tightly. Set the RC-Cor-net[®] to the desired pressure profile by turning the mouthpiece at the bent end. Enclose the mouthpiece of the RC-Cor-net[®] adapter with your lips, inhale and breathe in the aerosol and then exhale against the generated oscillating resis-tance.

Recommended frequency of the combination therapy: 2–3 x daily, 2–5 minutes (with severe mucus buildup more often).

RC-Cor-net[®] adapter with mouthpiece (ID 20,3 mm)

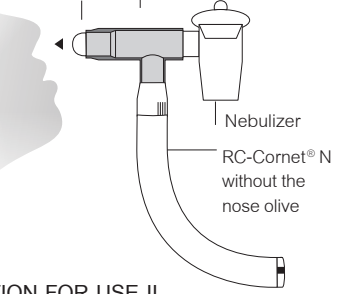INSTRUCTION FOR USE II

For the combination therapy of the upper respiratory tracts

The RC-Cor-net[®] adapter with mouthpiece combines the RC-Cor-net[®] N with all common nebulizers.

Pull the nose olive out of the RC-Cor-net[®] N. Place it into the RC-Cor-net[®] adapter and then add the converted adapter onto the RC-Cor-net[®]. Connect the system to the nebulizer, so that it fits tightly (see image). Caution: The mouthpiece of the nebulizer must first be removed.

Set the desired pressure profile (setting positions 0–1) on the RC-Cor-net[®] by turning the mouthpiece at the bent end.

Close off one nose hole with your pointer finger and push the nose olive of the RC-Cor-net[®] adapter into the other nose hole.

Breathe in the aerosol through the nose olive and generate the healing pressure oscillation in the nose-throat area by exhaling.

Recommended frequency of use of the combination therapy: 2–3 x daily, 2–5 minutes (with severe mucus buildup more often).

CLEANING RECOMMENDATIONS

The RC-Cor-net[®] adapter with mouthpiece can be cleaned as follows:

- In the RC-Clean[®] cleaning bag for the microwave (sold separately)
- Boiling
- In a vaporizer
- With hot steam: temperatures up to 134 °C

Clean the RC-Cor-net[®] adapter with mouthpiece every 2 days!

The RC-Cor-net[®] adapter with mouthpiece is meant for use by a single patient. For clinical reuse instructions for multiple patients, visit www.cegl.de

Please follow the instructions for use of the RC-Cor-net[®] / RC-Cor-net[®] N.

Contraindication RC-Cor-net[®] N:

The RC-Cor-net[®] N should not, or only after consultation with an attending doctor, be used when the following illnesses are present:

- Untreated pneumothorax
- Surgical operations or injuries in/on mouth, face, skull or esophagus
- Repeated, massive nose bleeding
- Acute nose-sinus or middle ear inflammations usage should only follow with concurrent administering of antibiotics and after consultation with a doctor

Contraindication RC-Cor-net[®]

Presence of pneumothorax, blood-expectorating cough (massive hemoptysis), and intracranial pressure symptoms > 20 mm Hg.

ADVICE: The RC-Cor-net[®] should only be used after discussing it with the doctor treating you if the following are present:

- severe cardiovascular disease
- acute thrombosis
- acute coughing up of blood (hemoptysis)
- surgery or injury to the mouth, face, skull, or esophagus
- possible eardrum injuries
- pulmonary barotrauma
- significant cardiovascular insufficiency
- massive restriction of the respiratory pump
- repeated, substantial nosebleeds

article-no. 225 | PPN 1101 4131 3578

Other products available in pharmacies and medical supply stores.

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