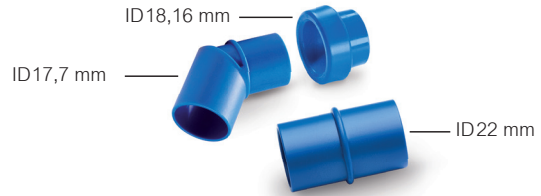
RC-Cornet[®] PLUS VERNEBLER SET

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
2651m | 07/2021

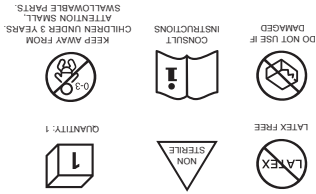LOT 210720

All trademarks and registered trademarks belong to the company CEGLA Medizintechnik GmbH & Co. KG, Germany. In some instances, product graphics may vary from actual product models, sizes and/or colors listed below.


CEGLA
MEDIZINTECHNIK

CEGLA Medizintechnik GmbH & Co. KG
Horresser Berg 1
56410 Montabaur
Germany
Tel +49 2602 92 13-0
info@cegla.de
www.cegla.de

Please read the instructions for use RC-Cornet[®] PLUS NEBULIZER KIT carefully before first use. Please also follow the instructions of the according RC-Cornet[®] PLUS respiratory physiotherapy devices, the nebulizer and the medication. All three products are included and must be purchased additionally. In combination with nebulizers, it is important to ensure that the RC-Cornet[®] PLUS with and without the RC-Cornet[®] PLUS NEBULIZER SET is as securely fitted as it would be if the nebulizer mouthpiece was attached. During inhalation you should not see any inhalation solution flowing outside of the product combination. To this end, pay attention to the respective interior diameter (ID).

Article no. 2651 | PPN 111 443 990 049
* Depending on the application deviations in the deposition data are possible.

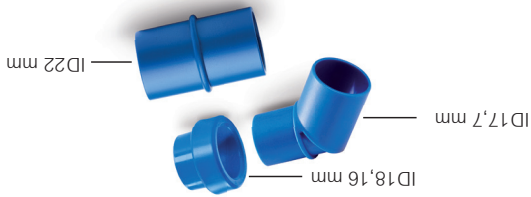
This RC-Cornet[®] NEBULIZER KIT package contains: Nebulizer ring, elbow connector and RC-adapter

RC-Cornet[®] PLUS NEBULIZER KIT enables a combination of the RC-Cornet[®] PLUS FAMILY with almost all non standard nebulizers.*

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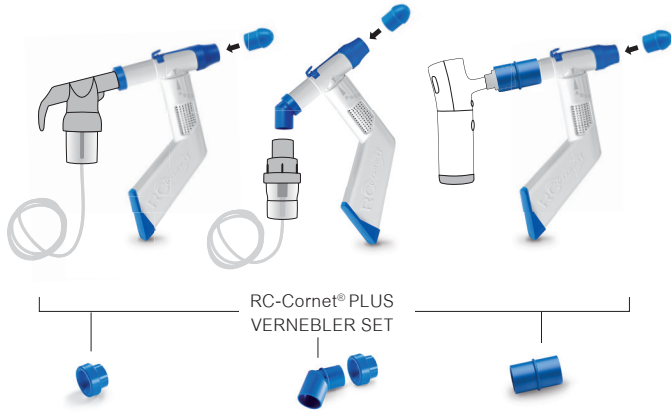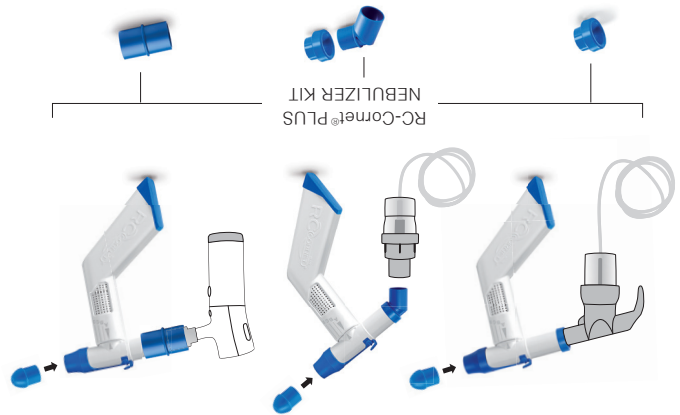
Material: Polypropylen

COMBINED THERAPY / NEBULIZER CONFIGURATION WITH THE RC-Cornet® PLUS FAMILY:

Remove the nebulizer mouthpiece. Then connect the nebulizer with one, two or all three parts of the RC-Cornet® PLUS NEBULIZER KIT (see illustrations above).
Now apply the RC-Cornet® PLUS FAMILY (RC-Cornet® PLUS, RC-Cornet® PLUS NASAL, RC-Cornet® PLUS TRACHEO) according to the characteristics in setting 1.

The integrated one way valve now enables an inhalation of aerosol once you have started your compressor. When you exhale, the valve closes and you start your respiratory physiotherapy via the RC-Cornet® PLUS FAMILY.

Please be aware that the nebulizer as mentioned in the instructions for use is always in an upright position.
TIPP: If you feel uneasy to use setting 1 at the beginning, you can use the middle setting between 1 and 2 to start with.

Recommended usage:
2-3x daily, or as recommended by your healthcare professional. The timing should be equivalent to the recommended timing going with the nebulizer or the drugs applied.

The same cleaning instructions apply for the RC-Cornet® PLUS NEBULIZER KIT as for the corresponding RC-Cornet® PLUS product.
Material: Polypropylen